



## Surgitex Multi Purpose Foot Protection & Booties

- Pressure control
- Foot support
- Protection and comfort



### INDICATIONS

*-INTRA-OPERATIVE AND POST-OPERATIVE PRESSURE RELIEF*

*-INTRA-OPERATIVE AND ORTHOPAEDIC USE INCLUDING TRACTION*

*-DIABETES AND PERIPHERAL NEUROPATHY*

*-PARALYSIS AND WHEEL-CHAIR PROTECTION WITH COMFORT*

*-SKIN CONDITIONS*

*-NEUROLOGICAL DISORDERS*

*-VASCULAR DISORDERS*

*-OBESITY CARE*

### •ORTHOPAEDIC TRACTION BOOTIES / PROTECTORS


- Versatile
- Compatible with majority of orthopaedic booties

### Key Features


- Ultra lightweight
- Seamless internally
- Washable
- Ease of use
- Compact
- Comfortable yet firm



## Surgitex Multi-Purpose Foot Protection & Booties

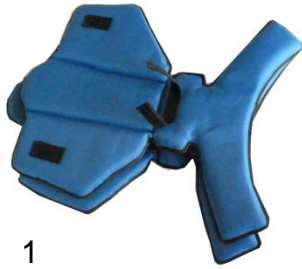

1

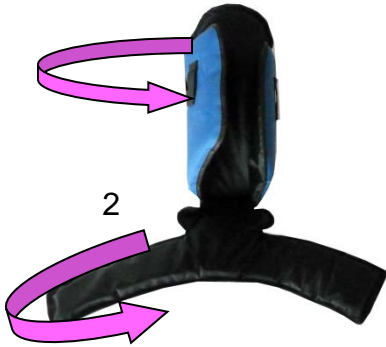

2



3

### User Instructions

- Open out the bootie pad to the foot shape with the black surface inside and blue outside (figure 1)
- Wrap the pad around the foot
- Use the straps to secure the pad around the foot, making sure the longer wings overlap the shorter wings, and there is no contact between the straps and the skin
- The straps should neither be too loose to be insecure nor too tight as to restrict circulation. The same precautions apply if bandages are used as additional or replacement strapping.
- Adjust straps to the desired tightness making sure that the tip of the greater toe is exposed

### Caution:

- Do not tighten the straps excessively
- Do not use the booties continuously or for the long-term
- For pressure, wound or surgical applications consult a clinical specialist
- Not designed for walking any distance on hard surfaces.

### Orthopaedic Traction Application

- 1- insert the pad inside the Orthopaedic traction boots
- 2- put the sole of the pad against foot plate of the Orthopaedic bootie (black inside)
- 3- Secure the foot inside the pad using the bootie straps provided, or a combination of these with bandages to secure the foot against the Orthopaedic Traction Plate (Fig.4&5)

### Ordering information

Size 35 to 43. Cat Number. S900-1 Colour light blue. Pair  
 Size 44.to 48. Cat Number. S900-2 Colour dark blue. Pair



4



5

Each Pair includes :  
 Two 120cm Straps  
 Two 50 cm Straps  
 Two 20 cm loops